



**INTEGRA  
TOOLS**

Formerly  
LIFEtool Solutions



VERSION  
// 1.0

# IntegraMouse<sup>®</sup> AIR Quick Guide

integramouse.com



English (EN)



**Commissioning,  
Cleaning & Controlling**

# Commissioning

## Mounting of IntegraMouse AIR

1.



Insert the mouthpiece, paying attention to guide pin and guide slot.

2.



Apply slight pressure and turn the mouthpiece anti-clockwise until it clicks into place.

3.



Attach the IMAIR to your mounting system (1/4" thread).

4.



Bring the IMAIR into the desired working position.

**For initial commissioning, we recommend operation with a cable connection.**

5.



Plug the cable into the USB-C socket at the rear end of the IMAIR.

6.




Plug the USB cable into the computer (USB-A socket).

7.



Briefly press the right lip button to switch on the IMAIR.

8.

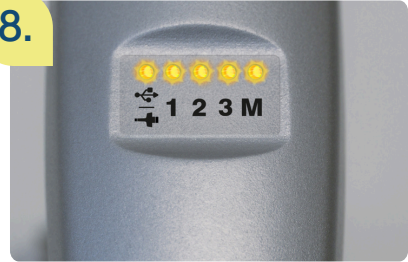

All LEDs light up briefly and the IMAIR calibrates itself automatically.

9.



The left LED lights up (cable connection).  
The right-hand LED (M) is off (mouse mode).



**Please do not  
touch the  
IntegraMouse  
AIR during  
calibration!**

**All further steps are executed automatically:**

- The internal battery is charging.
- Your computer installs the IntegraMouse AIR as the standard mouse.

# Quick Guide to Commissioning in Wireless Mode

(Pairing via Bluetooth)



To use the IntegraMouse AIR with Bluetooth, make sure that the battery is sufficiently charged. To do this, use the enclosed charger and charge the battery before using it for the first time.

1.



Briefly press the right lip button to switch on the IMAIR.

2.




All LEDs flash briefly and the IMAIR calibrates itself.

3.



The left LED lights up.

4.

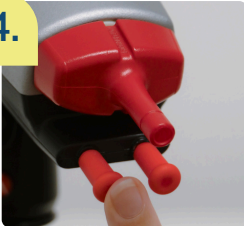

Press the left lip button for 3 seconds.

5.



LED 1 starts to flash.

6.




Select the desired position (1, 2 or 3) by pressing the left lip button.

7.



Wait briefly until the selected LED flashes blue.

8.

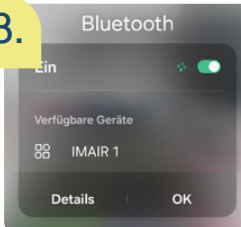

Start the device search on the end device and select IMAIR.

9.



If pairing is successful, the flashing stops and the LED lights up orange.

# General Notes on Cleaning



## Use of IntegraMouse AIR in the clinical sector

- In the clinical sector use the mouthpiece (including membrane and silicone tube) and the lip buttons as disposable products
- The interval between changes of the mouthpiece (including membrane and silicone tube) and lip buttons must be determined by a doctor or by specialist staff
- In case of suspected infectious diseases please take special safety precautions (e.g. wear gloves when changing the mouthpiece).

## Disinfection by wiping

- Use only surface disinfectants which are suitable for LEXAN polycarbonate.
- We recommend by Schülke & Mayr ([www.schuelke.com](http://www.schuelke.com)): Mikrozid Sensitive Liquid or Mikrozid Sensitive Wipes or equivalent products.



## Do not use wet or gamma sterilization

The mouthpiece, lip buttons and other components of the IntegraMouse AIR are not allowed to be cleaned with wet or gamma sterilization.

## Do not use aggressive cleaning agents

Never use aggressive cleaning agents, this will destroy the surface of the components

## Do not immerse in water

Never immerse the IntegraMouse AIR and their electronic components in water and never use running water – otherwise you could destroy the device



## Correct cleaning of components

- The parts of IntegraMouse AIR, power pack including cable and mounting system can be cleaned by wiping with a damp, lint-free cloth
- Clean the components once every two weeks

# CLEANING the mouthpiece & REPLACING the membrane

## Cleaning- and disposal-interval of the mouthpiece and lip buttons

**Cleaning:** Die IntegraMouse AIR has a reminder function for cleaning and reminds you after the recommended period of use with a flashing LED (see page 8).

- at least once every two weeks
- more often in case of illness

### **Dispose:**

- if damaged or signs of wear visible
- within six months

## **Disposal-interval of the membrane**

Always dispose of the membrane, no cleaning is recommended.



First turn the mouthpiece clockwise and then pull it straight off.



Remove and dispose of the membrane.




Clean the mouthpiece in a steam steriliser.



Leave mouthpiece to dry and cool.

5.

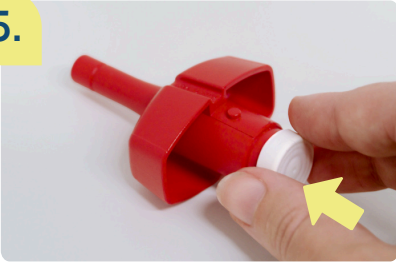

Place new membrane on mouthpiece.

6.



Insert the mouthpiece (note the guide pin and guide slot) and turn anti-clockwise with light pressure.














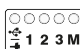
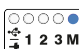



The mouthpiece can also be boiled in a saucepan with water.

- Bring water to the boil.
- Insert mouthpiece with a spoon and ensure that it is completely covered with water.
- Leave mouthpiece for at least 5 minutes in the boiling water.
- Carefully remove the mouthpiece from the saucepan with a spoon and leave it to dry and cool.

# Functionality of IntegraMouse AIR

The IntegraMouse AIR offers three operating modes, standard mode is mouse mode.

	Mouse	Keyboard	Gamepad
Move mouthpiece: <ul style="list-style-type: none"> <li>up</li> <li>down</li> <li>to the left</li> <li>to the right</li> </ul>			
Sip	Left click 	Space 	Button 1 
Puff	Right click 	Enter 	Button 2 
Microphone	Note: There is a microphone on the underside. This can only be used in cable mode.		
Calibrate: Press right lip button for 3 seconds	All LEDs flash simultaneously. ⓘ Please do not touch the mouthpiece.		
Change device: Briefly press left lip button	1x cable      3x Bluetooth    		
Change mode: Briefly press right lip button.	 Mouse	 Keyboard	 Gamepad
Reminder mouthpiece change	LED 0 flashes orange. Press both lip buttons simultaneously to reset the timer.		
Low battery warning	When the battery is low in wireless mode, the LEDs 1, 2 or 3 start to flash orange.		
Switch off: Press the right lip button for 6 seconds	Note: IMAIR must be active (LED illuminated). Press until all LEDs are off.		



For further setting options and functions, please download the free settings app from the IntegraMouse website at [www.integramouse.com](http://www.integramouse.com). There you will find further useful information and configurations.